

How to Install the GlucoPack™ System

Simple to Install



Step 1

Remove the existing battery cover from the wireless phone.



Step 2


Put the GlucoPack™ on the phone and push it slightly upwards.

How to Perform a Blood Glucose Test



Step 1

Flip the phone cover and press the down button to select "GlucoPhone" application to go to main menu.

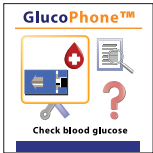

Step 2

At the main menu, select "Check blood glucose" by clicking OK.



Step 3


Insert the strip and perform the following steps:



a. Insert Strip

b. Lance your finger

c. Apply blood

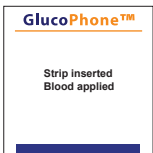

Step 4

Once blood is applied, the screen will automatically move on to the next step.



Step 5

Match the code number with the code number on the strip vial by pressing "up/down button" and select OK.



Step 6

Before you see the result, please select Yes/No.



If "Yes" was selected from the result screen, press OK after entering estimated carbohydrates.

Step 7

Please press Send button from the result screen to record the result, then the following will happen automatically.



Awaiting Result



Result



Sending the Result



Final screen

Step 8


After the test is done, click confirm to go back to the main menu to view your historical data and help menu.



View historical data



View numerical data

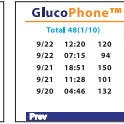

View



Help



Check blood glucose

Plug and Play



OK Options



Check blood glucose

Step 1

Open GlucoPhone™. Run GlucoPhone™ application to "Check blood glucose".

1. Insert GlucoPack™ Test Strip first and apply your blood to the top of the Test Strip.



Plug and Play



Strip inserted Blood applied

Step 2

Plug and Play Screen will be displayed shortly and moving to next screen. Please refer back to steps 5-8, above, to continue.

Error Message

1. Insecure installation of the GlucoPack™. Please try again.
2. Error - correction code must be verified. Please try again.
3. Blood injection is necessary. Please try again.
4. Strip error. Please use new strip.
5. Out of wireless network. Please try again.
6. Unauthorized user. Please register first and then use.

Result

Too Low Reading

The test result is lower than 10mg/dL

What to Do: Check the meter with the GlucoPack™ control solution. If a normal result occurs, re-test with blood sample two or three times. "Too Low" result indicates severe hypoglycemia (low blood glucose). If the "Too Low" message persists, contact your physician immediately.

Too High Reading

The test result is higher than 600mg/dL

What to Do: Check the meter with the GlucoPack™ control solution. If a normal result occurs, re-test with blood sample two or three times. "Too High" result indicates severe hyperglycemia (high blood glucose). If the "Too High" message persists, seek medical attention immediately.