


Installation

You will need to select a telephone jack that is near an electrical outlet. There are two options for connecting your Eocene cradle. Option one, connecting your Eocene cradle to a telephone jack where your phone is currently connected. Option two, connecting your Eocene cradle to a telephone jack where there is no phone connected.

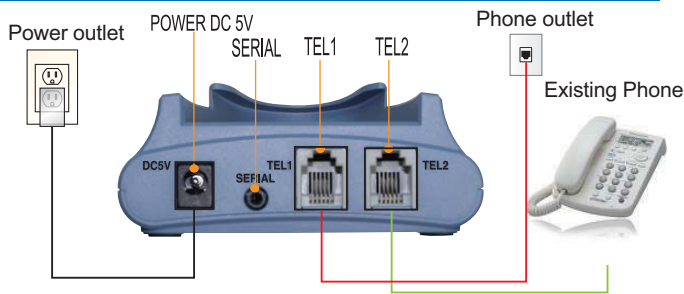

1. Unplug the telephone line from the wall jack and plug it into TEL2 on the back of your device.




2. Use the included phone cord and plug one end into the telephone jack. Plug the other into TEL1 on the back of your device. (**Note:** The main telephone line from the wall always goes to TEL1.)



3. Connect the included power cord into the port marked DC5V and the other end into your electrical outlet.



4. Your Eocene cradle is now ready to use.

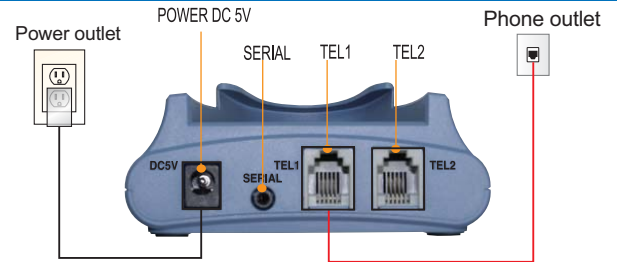

1. Use the included phone cord and plug one end into TEL1 on the back of your device. Plug the other end into the telephone jack.



2. Connect the included power cord into the port marked DC5V. Plug the other end into your electrical outlet.



3. Your Eocene cradle is now ready to use.

Please Note:

a. The Eocene Cradle will operate ONLY with a standard Analog Phone Line or POTS Line (Plain Ordinary Telephone Service). It will not work with a digital Phone Line, Cable TV, or VOIP Internet Service. This can be verified by contacting your telephone service provider.

b. If you have a computer and internet access, you can download the Eocene Software on your computer and use a USB Cable to upload your test results to the Eocene System. (USB Cable sold separately)