

Package contents

- 1) Eocene cradle 2) DC 5V Adaptor 3) Phone line 4) Owner's manual and quick reference guide



Please ensure your EOCENE™ system contains all of the items pictured. If any of the items are missing, please return the product to the original place of purchase or contact Infopia USA customer support at: [1-888-446-3246](tel:1-888-446-3246).

Eocene™ Comprehensive Diabetes Management & Compliance System

Remote Home Patient Monitoring System

EOCENE SYSTEMS

Quick Reference Guide

Manufactured Exclusively for
Infopia USA, LLC
P.O.Box 5532
Titusville, FL 32783
TEL: 888-446-3246
www.infopiausa.com


VIRTUAL TRACKER

Catalog number : GL-5C7-1B1-01

Operation



1. Insert your Infopia glucose meter face up into the Eocene cradle and press firmly until the meter "Stops". When properly inserted, the "Meter Light" on your device will light up.

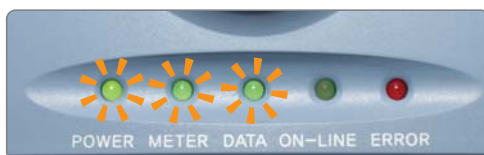

2. Once inserted the Eocene cradle "Data Light" will blink, this means the device is searching for new readings. If the "Data Light" goes off this means there are no new readings.



3. If your Eocene cradle finds new test results, the "Data Light" and "On-Line Light" will both light up. This means your test results are being transferred. (**NOTE:** Do not interrupt this process.)



4. Once the transmission is complete, the "Data Light" will shut off and will be followed by the "On-Line Light". It is now safe to remove your meter from your Eocene cradle.

Removing your meter from the Eocene Cradle.



5. Put your forefingers on the back of the device while holding the device in both hands, then use your thumbs to push the meter out as displayed.

Please remove your meter promptly from the cradle: Leaving your meter in the cradle will drain your Glucose meter battery!